Instructions for Use English from page 03

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Page 9 to 21 | Operation

How to use Daysy.

Page 22 to 30 | Technical notes and warranty

What else you need to know.

About Daysy

Daysy calculates a woman's daily menstrual cycle status by interpreting her basal body temperature. Daysy displays your menstrual cycle status using colored lights. Daysy is neither designed as a replacement for contraception, nor is it a medical thermometer.

Included in delivery:

- Daysy
- Sensor cap
- Micro USB cable
- Instructions for Use
- Quick Guide


— Precautions

- Before using your Daysy for the first time, please read these instructions carefully and keep them in a safe place for later reference.
- Before you start using your Daysy, you need to entirely discontinue hormonal methods of birth control.
- Daysy may have difficulty recognizing your infertile days if you have an irregular menstrual cycle.
- Daysy is not suitable for use shortly before the onset of menopause, if you are undergoing hormonal treatment, or if you are using hormonal methods of birth control.
- Daysy should only be used by women whose cycle is between 19 and 40 days.
- Daysy does not protect you from sexually transmitted diseases.
- You should always read your daily fertility status from Daysy itself, not the app.
- Before using Daysy for the first time, charge the battery fully.
- Never open the housing.
- Never chew on the sensor.
- Take care that the appliance or parts thereof are not swallowed.

- **WARNING:** The device must not be modified.
- **WARNING:** Avoid using Daysy at or less than 30 cm away from other electrical equipment, as this could result in improper operation.
- **WARNING:** Use of cables or equipment other than those specified in this document may cause electrical interference or lead to failure of Daysy.
- Do not expose your Daysy to high temperatures (e.g. boiling water).
- Do not expose your Daysy to direct sunlight for long periods of time.
- Only take measurements in your mouth.
- Do not insert the measuring sensor too far into your mouth.
- Do not take measurements if the housing is broken.
- Do not take measurements if you have an injury in your mouth.
- Do not expose your Daysy to heavy mechanical loads.
- Only take measurements when the USB cable is NOT connected.
- Only charge your Daysy using a standard 5V USB port (IEC 60950-1 compliant) with a maximum output current of 1A and a maximum power of 4.5W with a cable shorter than 50cm. Do not take measurements with your Daysy while it is charging.

- Do not stick sharp objects into your Daysy.
- If your Daysy causes irritation to your skin, we recommend you stop using it immediately and contact our Customer Support as soon as possible.
- All serious incidents associated with this product should be reported to the manufacturer and the relevant government authority responsible in the EU member state.
- Your Daysy is intended for you alone. You should not share it with other women. If you do wish to pass it on to a friend, you will need to delete your data first.
- Only give your Daysy to other persons complete with these instructions. You will also find the instructions and further useful information available for download at daysy.me.
- As a user, you can use every feature of your Daysy. You can also clean the sensor yourself when your Daysy is switched off.
- Your Daysy is not a contraceptive. Improper use may result in unwanted pregnancy.
- Keep Daysy away from children and pets.

All the features at a glance*

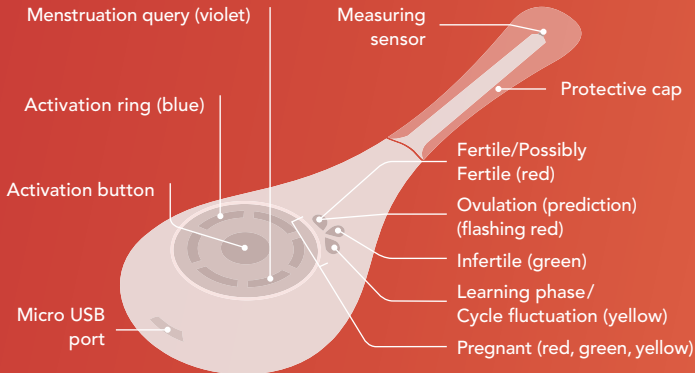* Page 13 shows you all the important display combinations

— Step 1: Charging and activating your Daysy

1. Connect your Daysy to the micro USB cable supplied in your package.
2. Connect the cable to a USB port. Make sure it is a 5V port that does not supply more than a 1A current.
3. As your Daysy charges, the blue light on the activation ring will move from the bottom upward. All the blue lights will light up once the battery is fully charged.
4. Disconnect the cable from your Daysy. You can now switch your Daysy on by briefly pressing the activation button.

Measurement indicators: What each indicator means

Violet light is flashing: Your Daysy is asking if you are menstruating. Confirm that you are menstruating by holding down the activation button for 2 seconds.

The violet light remains constantly illuminated: Daysy confirms your menstruation input. If you have entered your menstruation incorrectly or by mistake, hold the activation button for 2 seconds until the violet light starts flashing again.

Blue lights rotating in a clockwise direction: Your Daysy is measuring your temperature. Holding down the activation button for 2 seconds stops the measurement.

— Step 2: Measuring your temperature

1. Use your Daysy every morning before you get out of bed. To do so, remove the protective cap and briefly press the activation button once. Your predicted fertility status will be displayed.
2. Press the button a second time and the activation ring will flash in a clockwise direction. You can now take a measurement.
3. Place the sensor as far back as is comfortable under the right or left side of your tongue and then close your mouth.
4. "Beep-beep" means that the measurement phase has successfully completed.*
5. The confirmed fertility status displayed after measurement is valid for a maximum of 24 hours, or until you take the next measurement the following morning. You can view your fertility status again later in the day by pressing the activation button once briefly. The next day, your Daysy switches to the predicted fertility status before your measurement.

** If you hear a "grumble-buzz" sound instead of a "beep-beep", it means that the measurement was unsuccessful, and you will need to try again.*

Fertility indicators: What each indicator means

Fertile/Possibly Fertile

Pregnant

Infertile

Ovulating (prediction)

Learning phase/Cycle fluctuation


— Step 3: Entering your menstruation on your Daysy

If the violet **“Menstruation”** light is flashing in the activation ring, it means that Daysy is asking you for your menstruation input.

- **Menstruating? “Yes”:** Confirm you are menstruating by pressing the activation button until the violet light lights up without flashing and you hear a “beep-beep”. Confirm you are menstruating for each day of active bleeding and flow for a minimum of three consecutive days.
- **Menstruating? “No”:** You do not have to do anything; the violet light keeps flashing.
- **You are menstruating, but Daysy is not showing the flashing violet light:** Enter your menstruation input by holding down the activation button until the violet light is constantly lit.

Important: *The first time you start bleeding after you stop taking the pill, or stop using other hormonal methods of birth control, is referred to as hormonal withdrawal bleeding and does not mean that you are menstruating. Under no circumstances should you enter this bleeding as menstruation.*


DaysyView: Better informed with the Daysy app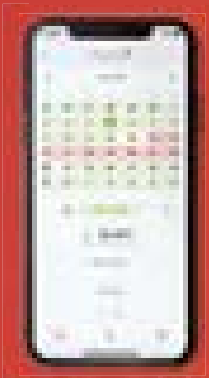

DaysyView gives you valuable information about your cycle. It also lets you share your information with your partner.

Turn on **Bluetooth®** on your mobile phone, open DaysyView, and press the Daysy activation button twice. A blue light will light up on your Daysy followed by a brief melody. If the blue light stays lit, it means the connection was successful. Afterwards, simply follow the instructions on the app.

Temperature and fertility: The course of your cycle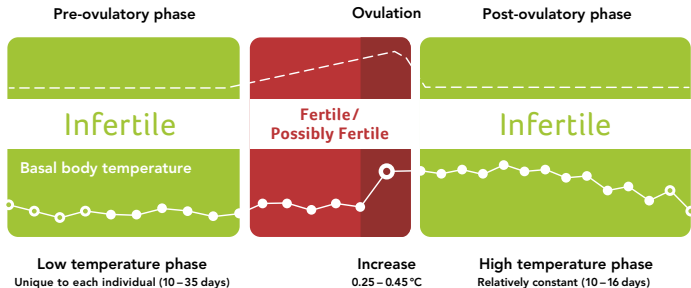

Important:

How to you use your Daysy correctly

- Make sure the first thing you do is take your temperature every morning before you get out of bed. If you forget, and have already gotten out of bed, do not proceed to take your temperature.
- If you are menstruating, enter this input accordingly.
- The more regularly you take your temperature, the faster Daysy will get to know your cycle.
- When you start using your Daysy, you will initially see a lot of yellow and red days. Over the course of the next 3 to 4 cycles, Daysy learns your unique menstrual cycle rhythm and you will see an increasing number of green days.

— Cleaning

- Clean Daysy before and after every use
- Use a damp cloth to carefully clean the sensor on your Daysy. You can also use a little alcohol or soap if you wish. Since your Daysy is not waterproof, make sure that only the neck on the sensor and not the whole appliance comes into contact with water.

Battery

- A single battery charge will last 1 to 2 months, depending on how often you use your Daysy. To indicate that the battery is nearly empty, the activation ring on your Daysy will flash at frequent intervals. Make sure you charge the battery within a week. With normal use, the battery will still charge to 80% of its capacity after 500 charging cycles.
- **Note:** You should not take measurements when your Daysy is charging.

Error messages/ Problems during use

My Daysy does not respond when I press the activation button.

Please charge your Daysy and try activating her again before contacting our Customer Support.

The activation ring on my Daysy flashes blue at short intervals.

Please charge the battery on your Daysy.

My Daysy fails to start taking a measurement.

- Remove the USB cable before taking a measurement.
- If your Daysy makes a “beep-beep” sound, it means you have already successfully taken a measurement today.

For more information, read our FAQ at daysy.me or contact our Customer Support.

My Daysy ends the measurement with a “grumble-buzz” instead of a “beep-beep”.

The measurement took too long, was interrupted, or was unsuccessful.

Try following these tips:

- As before, place the sensor as far back as is comfortable under the right or left side of your tongue.
- Close your mouth.
- Do not remove the your Daysy from your mouth until you hear the “beep-beep”.
- Daysy may not accept the measurement if you have a fever.
Stop taking measurements if you are sick.
- If the sensor gets dirty, simply clean it with a damp cloth.

The red indicator lights up and the activation ring flashes at short intervals.

Something is wrong with your Daysy. Please contact our Customer Support.

I won't need my Daysy in the next few months.

What do I need to do?

If you do not intend to use your Daysy for a long period, clean her, charge the battery fully, and store her in a cool, dry place.

How does my Daysy communicate with me?

- “Beep-beep”: Daysy is happy. The measurement was a success.
- “Grumble-buzz”: Daysy is unhappy. The measurement was not a success.
- “Beep”: Daysy beeps when you press the activation button.
- Melody: Daysy has just activated **Bluetooth®**.

For more information, read our FAQ at daysy.me or contact our Customer Support.

— Technical data

Key features

Name and model: Daysy **Power supply:** Internal power source

Contains FCC ID: WAP2001 **IC:** 7922A-2001 **USB port:** USB 2 Micro-B

Compatibility

iOS: Version 10.0 and higher **Android:** Google Android version 5.0 and higher

Bluetooth®: Version 4.0 and higher

Dimensions: H/W/D 13/2.2/4.8 cm **Weight:** 38g **EAN Code:** 7 629999 05407 3

Technical Information

You need a **Bluetooth®**-enabled smartphone or a tablet running iOS 10.0/Android 5.0 or higher to be able to use the DaysyView app.

Bluetooth®: Bluetooth® 4.1 **Frequency band:** 2400 – 2480 MHz

GFSK modulation: FHSS **Transmitting power:** 0 dBm

Indications and contraindications

Indications: Daysy is a menstrual cycle tracker used by women who have a regular 19 to 40-day cycle.

Contraindications: In the following cases, you should **not** use Daysy:

- You suffer from a condition in which pregnancy could be dangerous for you, e.g. your blood pressure is too high or too low, you are suffering from heart disease, etc.
- Health risks exist during pregnancy for your unborn child, e.g. if you are suffering from drug or alcohol addiction, or if you are taking medication that may cause birth defects.
- When there are signs of menopause, if you are undergoing hormonal treatment, or if you are using hormonal methods of birth control.
- In case of irregular bleeding. This can interfere with the prediction of your fertility or render it unusable.
- Daysy is not a medical thermometer and should not be used to measure body temperature as an indication of fever or illness.
- Daysy is not a contraception.

— Note on electromagnetic compatibility


Electrical medical appliances are subject to special precautions regarding electromagnetic compatibility (EMC). Daysy is intended for non-commercial home use only.

Portable and mobile RF communication appliances may affect medical electrical equipment. Daysy meets the requirements of EN 60601-1-2 for electromagnetic compatibility. Details of this measurement data can be obtained from our Customer Support or from daysy.me.

Target group profile

Women aged 18 or older, who have completed nine or more school years and can read and write.

Device Label

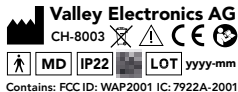The following figure shows the device label that is attached to the back of your Daysy. The label is shown enlarged for improved readability.

General information

Manufacturer

Read these instructions carefully

Attention

CE label

Medical device

**LOT number /
manufacturing date**

Patent number: US 8,998,822

WEEE: Please dispose of this appliance at public recycling depots for electrical appliances. Local laws/regulations prohibit disposing of this appliance and/or its battery with your general domestic garbage.

Type-BF application parts

IP code: Protected against solid foreign bodies with a diameter of 12.5 mm and splash-proof when the housing is tilted at 15°.

Storage and operating conditions

Temperature limit: This device can be stored at temperatures between $-20\text{ }^{\circ}\text{C}$ and $45\text{ }^{\circ}\text{C}$.

Temperature limit: This device can be used without restriction at temperatures between $5\text{ }^{\circ}\text{C}$ and $40\text{ }^{\circ}\text{C}$.

Humidity limit: This device can be stored without restriction at a relative humidity up to 93 %.

Humidity limit This device can be used without restriction at a relative humidity of between 15 % and 93 %.

Pressure limit: This device can be stored at pressures of between 700 hPa and 1060 hPa.

Pressure limit: This device can be subjected to pressures of between 700 hPa and 1060 hPa.

— Warranty information

Daysy has been manufactured with the utmost care using premium-quality, state-of-the-art materials. Each Daysy is subjected to numerous controls and inspections prior to delivery.

1. We guarantee the original purchaser that Daysy is free of defects in materials and workmanship during the warranty period.
2. The warranty period begins on the day the new Daysy is sold to the end user by an authorized dealer.* The warranty is valid for two years.
3. We undertake to either repair a defective device or replace it with a new one. Further claims, and, in particular, claims for losses and damages, including consequential damages, are excluded. The warranty does not cover indirect damages.

** Please note that our limited warranty applies only if you purchased your Daysy from an authorized Valley Electronics dealer.*

4. The warranty does not cover any damage caused by improper handling, especially by opening the device, damage to the inspection sticker, or by other external influences.
5. Warranty claims are only valid if the user indicates the defect immediately in writing within two weeks after such a defect is discovered. Once the defect is discovered, the user should no longer use the Daysy, but return it to the authorized distributor with a copy of the original purchase invoice via insured delivery.

If you have any questions or require further information, please do not hesitate to contact your Daysy dealer.

— Overview: What do the indicators mean?

Status Indicator			M light	Ring	Meaning
Red	Green	Yellow	Violet	Blue	
Illuminated	-	-	-	-	Fertile/Possibly Fertile
Flashes	-	-	-	-	Ovulating (prediction)
-	Illuminated	-	-	-	Infertile
-	-	Illuminated	-	-	Learning phase or cycle fluctuation
Illuminated	Illuminated	Illuminated	-	-	Pregnancy
Flashes	Flashes	Flashes	-	-	Possible pregnancy
-	-	-	Flashes	-	Daisy asks for your menstruation input
-	-	-	Illuminated	-	Menstruation confirmed for this day
-	-	-	-	Rotates	Temperature measurement
-	-	-	-	Flashes	Charge battery
-	-	-	-	Rising	Battery charging
-	-	-	-	Illuminated	Battery charged
-	-	-	-	1 light flashes	Bluetooth® enabled

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Valley Electronics AG
Valley Electronics LLC